

Aluminum alloy manual wheelchair - [light and comfortable]

鋁合金手動輪椅車-【輕便舒適款】

Model: LR-WBN

Features:

- Ergonomic design, seat tilt back 8°, anti front sliding
- The whole vehicle can be folded with one button for fast storage
- The movable armrest turns 360°, which makes it more convenient to get on and off wheelchairs
- Integrated pedal, dual brake system
- Double layer cushion detachable cleaning
- Suitable for people with muscle weakness and mobility difficulties
- 人體工學設計, 座椅後傾8°, 防前滑護脊椎
- 整車一鍵收折, 便於快速收納
- 活動扶手360°翻轉, 上下輪椅更方便
- 一體式腳踏板, 雙剎車系統
- 雙層坐墊可拆卸清洗
- 適合人群: 肌無力, 行動不便的人



Specifications

Product Model 產品型號	LR-WBN
Material 材質	aluminum alloy 鋁合金
Product Function 產品功能	out walking aid, foldable 外出助行, 可折疊
Control Mode 控制方式	manual control 手動控制
Front tyre 前輪尺寸	6 inches
Rear tyre 後輪尺寸	12 inches
Product Size 產品尺寸	102*56*124cm
Product Folding Size 折疊尺寸	58*40*88cm
Product Weight 產品重量	10.5kg

Instructions

◆輪椅展開和折疊方法:

1. 雙手握住把手, 將車後輪提起離開地面, 向兩側輕拉
2. 將左右車架分開(靠墊上沿繃直), 向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平, 系好腿帶
4. 用手提拉坐墊中間, 使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

◆注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中, 以免受傷
2. 切勿在未剎住駐立剎車時乘坐輪椅, 以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上輪椅, 謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅, 以免造成輪椅損壞或身體傷害
5. 下坡時, 不能使用駐立剎車, 以免翻車造成人身傷害
6. 下車時請務必先剎住駐立剎車, 駐立剎車是為使用者上下輪椅或停靠時使用, 不能用來在行駛中作制動使用