

Folding portable full-automatic electric Wheelchair [high allocation]

折疊輕便全自動電動輪椅-【高配款】

Model: LR-WBP

Features:

- High strength alloy frame, bearing 100kg
- New lithium battery, can be carried on the plane. Super long endurance 20km
- 360 degree universal control, simple and easy to operate
- One button folding, easy storage, adjustable armrest operation
- Manual / electric, one key switching, multi gear speed adjustable
- Front and rear dual control, free switching, self and nursing staff can operate
- Suitable for people with muscle weakness and mobility difficulties
- 高強度合金車架, 承重100kg
- 新型鋰電池, 可攜帶上飛機, 超長續航20KM
- 360度萬向控制, 簡單易操控
- 一鍵折疊, 收納輕鬆, 扶手操作可調節
- 手動/電動, 一鍵切換, 多檔速度可調節
- 前後雙控, 切換自由, 自己和護理人員均可操作
- 適合人群: 肌無力, 行動不便的人

Specifications

Product Model 產品型號	LR-WBP
Material 材質	aluminum alloy 鋁合金
Product Function 產品功能	out walking aid, foldable 外出助行, 可折疊
Control Mode 控制方式	manual control + electric control 手動控制+電動控制
Front tyre 前輪尺寸	8 inches
Rear tyre 後輪尺寸	12 inches
Product Size 產品尺寸	96*62*86cm
Product Folding Size 折疊尺寸	61 * 36 * 90cm
Product Weight 產品重量	17kg



Instructions

◆使用說明:

- 1.將腳踏板向上翻起, 坐上電動輪椅車後, 再將腳踏板放下
- 2.坐上電動輪椅車後, 請系好安全帶, 打開電源, 推動操縱杆, 電動輪椅車啟動行駛
- 3.操縱杆可同時控制方向和速度,
- 4.若要使電動輪椅車停止, 只要鬆開手, 使得操縱杆復位即可
- 5.控制器的速度調節按鈕可調節電動輪椅車的速度
- 6.手推電動輪椅車時, 先把控制手柄開機再把總電源開關關閉就可以輕鬆推動電動輪椅車
- 7.下車前, 先關閉控制器電源, 壓下駐力剎車, 然後將腳踏板向上翻起

◆注意事項:

- 1.在閱讀和完全理解說明書前, 切勿操作您的電動輪椅車
- 2.建議精神異常者, 反應遲鈍者、操縱有困難者勿使用本電動輪椅車