

Steel tube manual multifunctional Nursing wheelchair

鋼管手動多功能護理輪椅

Model: LR-WBM

Features:

- High backrest wheelchair with 180 ° angle adjustment
- The hanging feet can be folded to the rear side to get on and off safely
- Lengthened armrest, height adjustable, seat widened and deepened
- Front and rear double brakes, removable dining table board, wheelchair with pull-out bedpan
- The full lying model is suitable for people with damaged lumbar spine, inconvenient limbs, hemiplegia, sleepiness and so on

- 可全躺設計高靠背輪椅, 可180°調節角度
- 掛腳可收折至後側方, 安全上下車
- 加長扶手, 高低可調節, 座位加寬加深, 加固車架
- 前後雙剎車, 更安全
- 餐桌板可拆卸, 輪椅含抽拉式坐便盆
- 全躺款適合: 腰椎受損、四肢不便、偏癱、嗜睡等人群

Specifications

Product Model 產品型號	LR-WBM
Material 材質	Steel 鋼
Product Function 產品功能	out walking aid, foldable 外出助行, 可折疊
Control Mode 控制方式	manual control 手動控制
Front tyre 前輪尺寸	8 inches
Rear tyre 後輪尺寸	24 inches
Product Size 產品尺寸	130*68*128cm
Product Folding Size 折疊尺寸	96*31*104cm
Product Weight 產品重量	33kg



Instructions

◆ 輪椅展開和折疊方法:

1. 雙手握住把手, 將車後輪提起離開地面, 向兩側輕拉
2. 將左右車架分開(靠墊上沿繃直), 向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平, 系好腿帶
4. 用手提拉坐墊中間, 使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

◆ 注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中, 以免受傷
2. 切勿在未剎住駐立剎車時乘坐輪椅, 以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上下輪椅, 謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅, 以免造成輪椅損壞或身體傷害
5. 下坡時, 不能使用駐立剎車, 以免翻車造成人身傷害
6. 下車時請務必先剎住駐立剎車, 駐立剎車是為使用者上下輪椅或停靠時使用, 不能用來在行駛中作制動使用