

# Aluminum alloy manual multifunctional Nursing wheelchair

## 鋁合金手動多功能護理輪椅


Model: LR-WBL

### Features:

- High backrest wheelchair with 180 ° angle adjustment
- The hanging feet can be folded to the rear side to get on and off safely
- Lengthened armrest, height adjustable, seat widened and deepened
- Front and rear double brakes, removable dining table board, wheelchair with pull-out bedpan
- Aluminum alloy, light, strong and durable
- The full lying model is suitable for people with damaged lumbar spine, inconvenient limbs, hemiplegia, sleepiness and so on
- 可全躺設計高靠背輪椅, 可180°調節角度
- 掛腳可收折至後側方, 安全上下車
- 加長扶手, 高低可調節, 座位加寬加深, 加固車架
- 前後雙剎車, 更安全
- 餐桌板可拆卸, 輪椅含抽拉式坐便盆
- 鋁合金材質, 輕便·堅固·耐用
- 全躺款適合: 腰椎受損、四肢不便、偏癱、嗜睡等人群

### Specifications

Product Model 產品型號	LR-WBL
Material 材質	aluminum alloy 鋁合金
Product Function 產品功能	out walking aid, foldable 外出助行, 可折疊
Control Mode 控制方式	manual control 手動控制
Bearing Capacity 承重	100kg
Lying size 躺平尺寸	148cm
Rear tyre 後輪尺寸	22 inches
Product Size 產品尺寸	128*72*94cm
Product Folding Size 折疊尺寸	89*33*99cm
Product Weight 產品重量	29kg

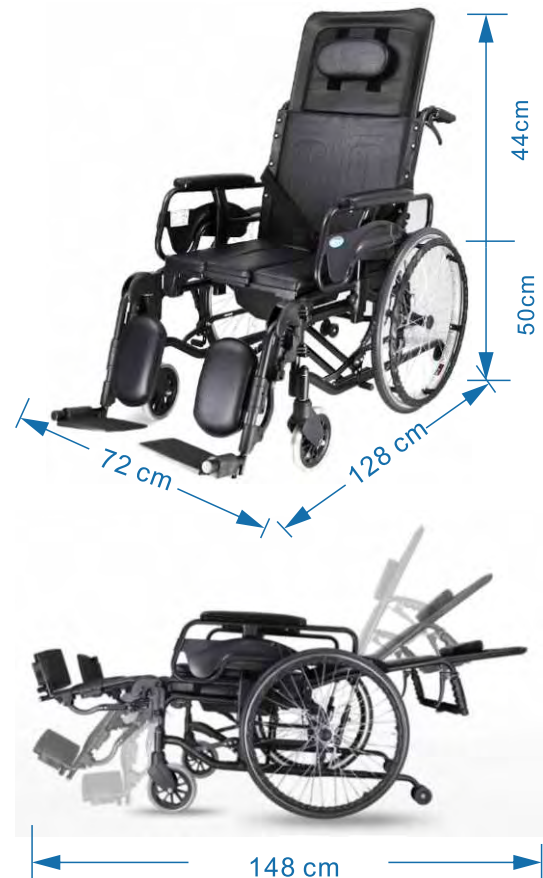

### Instructions

#### ◆輪椅展開和折疊方法:

1. 雙手握住把手, 將車後輪提起離開地面, 向兩側輕拉
2. 將左右車架分開(靠墊上沿繃直), 向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平, 系好腿帶
4. 用手提拉坐墊中間, 使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

#### ◆注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中, 以免受傷
2. 切勿在未刹住駐立剎車時乘坐輪椅, 以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上輪椅, 謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅, 以免造成輪椅損壞或身體傷害
5. 下坡時, 不能使用駐立剎車, 以免翻車造成人身傷害
6. 下車時請務必先刹住駐立剎車, 駐立剎車是為使用者上下輪椅或停靠時使用, 不能用來在行駛中作制動使用