

Aluminum Alloy Manual Wheelchair Portable

鋁合金手動輪椅輕便便攜


Model: LR-WAW

Features:

- Widened seat design, fast folding in 5 seconds, more convenient travel
- Lightweight and compact, there are 4 brakes at the front and rear, so you can use it with confidence
- Concave-convex non-slip armrest ring, easy to use
- Convex shape more non-slip
- Oxford fabric is easy to clean and easy to take care of
- The aluminum alloy frame is sturdy and durable, and the dual-support frame has strong stability
- Suitable for people with muscle weakness and mobility difficulties
- 加寬座位設計, 輕便小巧前後共4剎車, 方便護理, 放心使用
- 快速折疊, 只為出行更便捷
- 凹凸防滑扶手圈, 使用方便凹凸外形更防滑
- 牛津布料方便清洗, 易打理
- 鋁合金車架堅固耐用, 雙支持車架穩固性強
- 適合人群: 肌無力, 行動不便的人

Specifications

Product Model 產品型號	LR-WAW
Material 材質	aluminum alloy + oxford cloth 鋁合金+牛津布
Product Function 產品功能	out walking aid, foldable and light 外出助行, 可折疊輕便
Control Mode 控制方式	manual control 手動控制
Front Tyre 前輪尺寸	6 inches
Rear Tyre 後輪尺寸	20 inches
Net Body Weight 車身淨重	11.5kg
Product Folding Size 折疊尺寸	27*66*72cm
Package Size 包裝尺寸	30*73*74cm
Packing Weight 包裝重量	13kg

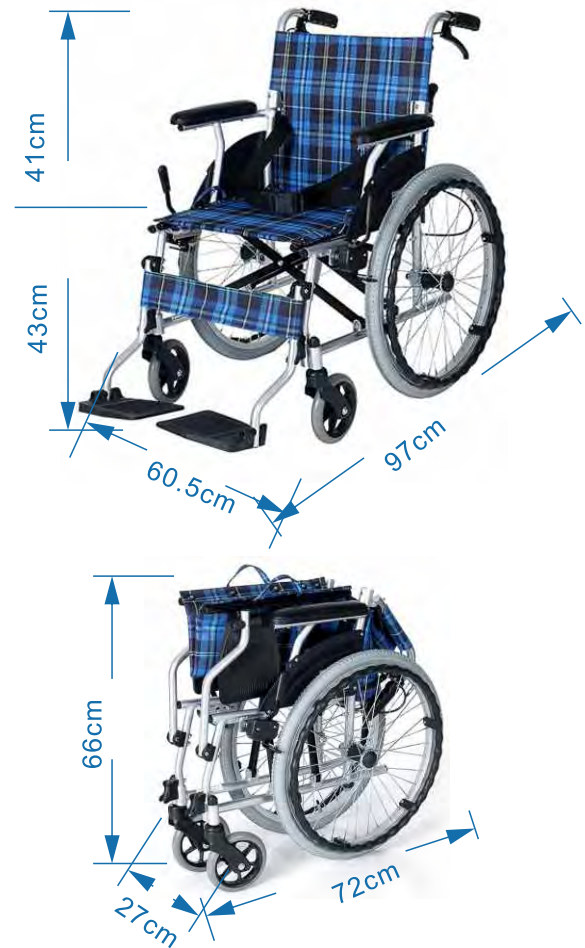

Instructions

◆輪椅展開和折疊方法:

1. 雙手握住把手, 將車後輪提起離開地面, 向兩側輕拉
2. 將左右車架分開(靠墊上沿繃直), 向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平, 系好腿帶
4. 用手提拉坐墊中間, 使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

◆注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中, 以免受傷
2. 切勿在未剎住駐立剎車時乘坐輪椅, 以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上輪椅, 謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅, 以免造成輪椅損壞或身體傷害
5. 下坡時, 不能使用駐立剎車, 以免翻車造成人身傷害
6. 下車時請務必先剎住駐立剎車, 駐立剎車是為使用者上下輪椅或停靠時使用, 不能用來在行駛中作制動使用